

zuverlässe®
Hand in hand with hygiene
Deutsch Technologie

BS PEROXY® RTU

HIGH LEVEL DISINFECTANT READY TO USE SPRAY

HAVING EN CLAIMS:

European Norms EN-17126:2018 (E) Sporicidal for medical area

European Norms EN-13727:2012+A2: 2015 (E) Bactericidal for medical area

European Norms EN-1275:2005 (E) Fungicidal & Yeasticidal for medical area.

European Norms EN-14476:2013+A2:2019 (E) Virucidal for medical area

European Norms EN-17387:2021 Bactericidal & Yeasticidal & Fungicidal for medical area.

European Norms EN-13624:2021 (E) Fungicidal & Yeasticidal for medical area.

European Norms EN-14563:2008 (E) Mycobactericidal & Tuberculocidal for medical area.

European Norms EN-17272:2020 Bactericidal, Mycobactericidal, Sporicidal, Fungicidal, Yeasticidal, Virucidal for medical area.



CDC
APPROVED



NOW WITH (EHP)
TECHNOLOGY

DESCRIPTION:

BS PEROXY® is a next-generation Escalated Hydrogen Peroxide (EHP™) Technology based surface and environmental chemical disinfectant, formulated in compliance with CDC guidelines to ensure the highest standards of safety and efficacy. It is completely non-toxic, non-carcinogenic, non-irritant, and eco-friendly, making it safe for routine use in critical healthcare and industrial environments. BS PEROXY® delivers rapid and reliable terminal disinfection of sensitive areas such as Operation Theatres (OT), ICCU, ICU, NICU, and recovery rooms, ensuring superior infection control. It is also the preferred choice for Quality Assurance (QA) and Quality Control (QC) sterile areas in the pharmaceutical and bioproduct industries, where uncompromising sterility and regulatory compliance are essential.

READY TO USE SPRAY:

Spray on the surface which needed to be disinfected and wait for 1-3 min to achieve claims, for sporicidal claim wait for 5 min, then wipe it with clean or sterile cloth.

APPLICATION AREAS:

ICU, CCU, Wards, OTs, Dialysis clinics emergency rooms home healthcare, hospitals, isolation areas, nursing rooms, outpatient, clinics, operating rooms, patient rooms. This product does not leave corrosive residue and may be used on stainless steel, hard plastics, nonporous vinyl, painted surfaces, Laminated plastic counter tops, fibre glass (sealed) surfaces, glazed porcelain, glass and all types of Ultrasound Probes including (Ultrasound Probe (General), Transvaginal Probe, Transrectal (TRUS) Probe, Intraoperative Probe, Laparoscopic Probe, Cardiac Probe), Operating table, Anesthesia machine, Surgical lights, Patient monitors, Ventilator, Patient beds, Pediatric ventilator, MRI scanner, CT scanner, Biosafety cabinets etc.



BACTERIA
/SPORES



NON-
TOXIC



SKIN
FRIENDLY

Zuverlässe GmbH
Nürnberg, Germany.
management@zuverlassegmbh.de



www.zuverlasseindia.com



www.zuverlasse.solutions



Zuverlasse Hygiene India Pvt. Ltd.
Plot No. F4, B/3, IDA, Kukatpally, Gandhi Nagar,
Hyderabad, Telangana – 500 037.
contact@zuverlasse.com
sales@zuverlasse.com

Toll Free Number: 1800-258-6457



MATERIAL COMPATIBILITY

BS PEROXY® SPRAY is compatible to the following material:

- Silicon
- Polyvinyl chloride (PVC)
- Polycarbonate (PC)
- Ceramic film
- Glass
- Polyoxymethylene (POM)
- Acrylonitrile butadiene styrene (ABS)
- Polypropylene (PP)
- Polymethyl methacrylate (PMMA)
- Stainless Steel 316
- Aluminium

Compatibility Method: The test item was evaluated for 14 days (±2 h) against multiple test specimens. Visual and tactile inspections confirmed that the following specimens showed no significant changes after exposure to BS Peroxy®, in accordance with the contact test described in ISO 21530:2004.

COMPOSITION:

Hydrogen Peroxide 7.35%
Peracetic Acid 0.23%
Inactive Ingredients 92.42%

PACK SIZE:

500 ml (SKU 5151)



BROAD-SPECTRUM
EFFICACY



FAST-ACTION
DISINFECTION



ENVIRONMENT
FRIENDLY



TRUSTED
PROTECTION

Zuverlāsse GmbH
Nürnberg, Germany.
management@zuverlāssegmbh.de



www.zuverlāsseindia.com



www.zuverlāsse.solutions



Zuverlāsse Hygiene India Pvt. Ltd.
Plot No. F4, B/3, IDA, Kukatpally, Gandhi Nagar,
Hyderabad, Telangana – 500 037.
contact@zuverlāsse.com
sales@zuverlāsse.com

Toll Free Number: 1800-258-6457