

HYDRO PRO™

HIGH LEVEL CHEMICAL DISINFECTANT FOR SURFACE & ENVIRONMENT

zuverlasse®
Hand in hand with hygiene
Deutsch Technologie



HAVING EN CLAIMS:

European Norms EN-17126:2018 (E) Sporicidal for medical area
European Norms EN-13727:2012+A2: 2015 (E) Bactericidal for medical area
European Norms EN-1275:2005 (E) Fungicidal & Yeasticidal for medical area.
European Norms EN-14476:2013+A2:2019 (E) Virucidal for medical area
European Norms EN-17387:2021 Bactericidal & Yeasticidal & Fungicidal for medical area.
European Norms EN-13624:2021 (E) Fungicidal & Yeasticidal for medical area.
European Norms EN-14563:2008 (E) Mycobactericidal & Tuberculocidal for medical area.
European Norms EN-17272:2020 Bactericidal, Mycobactericidal, Sporicidal, Fungicidal, Yeasticidal, Virucidal for medical area.

DESCRIPTION:

HYDRO PRO™ is new generation High Level Chemical Disinfectant based on Escalated Hydrogen Peroxide (EHP™) Technology this formulation designed for prolonged efficacy. It is a surface and environmental chemical disinfectant ensuring superior disinfection standards. It is non-toxic, non-carcinogenic, non-irritating, and eco-friendly at user dilution and it comes with inbuilt corrosion inhibitors. HYDRO PRO™ provides rapid terminal disinfection for critical and semi-critical areas.

APPLICATION AREAS:

Intensive Care Units (ICCU), Neonatal Intensive Care Units (NICU), and Recovery Rooms. CCU, Wards, OPD's, OT, Cath Lab, Cardiac OT, Radiology & Ultrasound, Obstetrics & Gynecology, Urology, ENT, Gastroenterology, Pulmonology, Dentistry, IVF Centre, CSSD, Oncology, Day Care Procedure Rooms, Dialysis clinics, emergency rooms, home healthcare, hospitals, isolation areas, nursing rooms, outpatient, clinics, operating rooms, patient restrooms, nursing stations etc.

RECOMMENDED DILUTION:

Mopping (Semi-Critical Areas) :

25 ml in 975 ml of DM water (Contact time 20-30 mins).

Mopping (Critical Area) :

50 ml in 950 ml of DM water (Contact time 20-30 mins).

Fogging in (Critical & Semi-Critical Areas):

50 ml in 950 ml of DM water (Contact time 20-30 mins).

COMPOSITION:

Hydrogen Peroxide 6 %
Inactive Ingredients 94 %



BROAD-SPECTRUM
EFFICACY



FAST-ACTION
DISINFECTION



ENVIRONMENT
FRIENDLY



TRUSTED
PROTECTION



Bacteria/
Virus



NON-
TOXIC



SKIN
FRIENDLY

COMPATABILITY:

Hydro Pro™ is compatible to the following material:

- Silicon
- Polyvinyl chloride (PVC)
- Polycarbonate (PC)
- Ceramic film
- Glass
- Polyoxymethylene (POM)
- Acrylonitrile butadiene styrene (ABS)
- Polypropylene (PP)
- Polymethyl methacrylate (PMMA)
- Stainless Steel 316
- Aluminium

Compatibility Method: The test item was evaluated for 14 days (± 2 h) against multiple test specimens. Visual and tactile inspections confirmed that the following specimens showed no significant changes after exposure to Hydro Pro™, in accordance with the contact test described in ISO 21530:2004.

PACK SIZE:

- 100 ml (SKU 7202)
- 500 ml (SKU 7201)
- 1 Liter (SKU 7200)
- 2 Liter (SKU 7203)
- 5 Liter (SKU 7204)



Zuverlasse GmbH
Nürnberg, Germany.
management@zuverlassegmbh.de
Toll Free Number: 1800-258-6457



www.zuverlasseindia.com



www.zuverlasse.solutions



Zuverlasse Hygeine India Pvt. Ltd.
Plot No. F4, B/3, IDA, Kukatpally, Gandhi Nagar,
Hyderabad, Telangana – 500 037.
contact@zuverlasse.com
sales@zuverlasse.com